

## COVID-19 Booster Shots: Moderna

- Designate and store your COVID-19 Moderna vials as booster shots with the following 2 barcode cards: one for the fridge, one for the freezer.
- Booster dose of the Moderna COVID-19 Vaccine is 0.25 mL (half of primary dose).
- Load either the 14-dose or 10-dose Moderna vials as a Booster shot, and they will be tracked to 20 doses each. We will track the new 20-puncture limit for you!

Any questions? Contact us 844-878-6331 x2 or [support@accuvax.com](mailto:support@accuvax.com)

## Moderna BOOSTER – 0.25mL

- Load as new vials
- Scan code
- Confirm **“BOOSTER”** in name
- Complete load
- Dispense by selecting **“BOOSTER”** card



**0.25mL**  
**20 Doses/Vial**

20 doses/vial follows FDA  
puncture limit guidelines

**Moderna BOOSTER – 0.25ML (FROZEN)**

Store **BOOSTER** in **FREEZER**

- Load as new vials
- Scan code
- Confirm "**BOOSTER**" in name
- Complete load
- Dispense by selecting "**BOOSTER**" card



**0.25mL**  
**20 Doses/Vial**

20 doses/vial follows FDA  
puncture limit guidelines

Store in REFRIGERATOR (2-8°C)

COVID-19 vaccine: **Pfizer**

Store **PFIZER** in **FRIDGE**

- Scan barcode
- Confirm “FRIDGE” in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in fridge



59267-0078-4  
10 per pack

Store in REFRIGERATOR (2-8°C)

COVID-19 vaccine: **Pfizer**

Store **PFIZER** in **FRIDGE**

- Scan barcode
- Confirm “FRIDGE” in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in fridge



59267-0078-2  
195 per pack

Store in REFRIGERATOR (2-8°C)

COVID-19 vaccine: **Pfizer**

Store **PFIZER** in **FRIDGE**

- Scan barcode
- Confirm “FRIDGE” in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in fridge



59267-1025-3  
25 per pack

Store in REFRIGERATOR (2-8°C)

COVID-19 vaccine: **Pfizer**

Store **PFIZER** in **FRIDGE**

- Scan barcode
- Confirm “FRIDGE” in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in fridge



59267-1025-4  
10 per pack

Store in REFRIGERATOR (2-8°C)

COVID-19 vaccine: **Pfizer**

Store **PFIZER** in **FRIDGE**

- Scan barcode
- Confirm “FRIDGE” in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in fridge



59267-1025-2  
195 per pack

**Store in REFRIGERATOR (2-8°C)**

COVID-19 vaccine: **Pfizer**

Store **PFIZER** in **FRIDGE**

- Scan barcode
- Confirm “FRIDGE” in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in fridge



59267-1055-4  
10 per pack

**Store in REFRIGERATOR (2-8°C)**

COVID-19 vaccine: **Pfizer**

Store **PFIZER** in **FRIDGE**

- Scan barcode
- Confirm “FRIDGE” in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in fridge



Store in FREEZER (-15°C)

COVID-19 vaccine: **Moderna (14-dose)**

Store **MODERNA (14-dose)** in  
**FREEZER**

- Scan barcode
- Confirm “FREEZER” in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in fridge



**Store in ULTRA LOW FREEZER (-70°C)**

COVID-19 vaccine: **Pfizer**

Store **PFIZER** in **ULT FREEZER**

- Scan barcode
- Confirm “ULTRA LOW” in name
- Manually enter lot & expiration
- Complete receive
- Place in Ultra Low Freezer



U5926710002

195 per pack

**Store in REFRIGERATOR (2-8°C)**

COVID-19 vaccine: **Moderna (14-dose)**

Store **MODERNA (14-dose)** in  
**FRIDGE**

- Scan barcode
- Confirm “FRIDGE” in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in fridge



**Store in ULTRA LOW FREEZER (-70°C)**

COVID-19 vaccine: **Pfizer**

Store **PFIZER** in **ULT FREEZER**

- Scan barcode
- Confirm “ULTRA LOW” in name
- Manually enter lot & expiration
- Complete receive
- Place in Ultra Low Freezer



U5926710003

25 per pack

**Store in REFRIGERATOR (2-8°C)**

COVID-19 vaccine: **Pfizer**

Store **PFIZER** in **FRIDGE**

- Scan barcode
- Confirm "FRIDGE" in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in fridge



**Store in REFRIGERATOR (2-8°C)**

COVID-19 vaccine: **Pfizer**

Store **PFIZER** in **FRIDGE**

- Scan barcode
- Confirm “FRIDGE” in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in fridge



25 per pack

**Store in FREEZER (-15°C)**

COVID-19 vaccine: **Moderna (10-dose)**

Store **MODERNA (10-dose)** in  
**FREEZER**

- Scan barcode
- Confirm “FREEZER” in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in freezer



## Store COVID-19 vaccines in AccuVax Units

AccuVax units can store and track all COVID-19 vaccines

When scanning original COVID-19 vaccine packaging the systems will **always default to FRIDGE**

Need to store in freezer or ultra low freezer? Use the attached barcode cards and follow steps, this will correctly store the COVID19 vaccine brand in the appropriate location

Any questions? Contact us 844-878-6331 x2 or [support@accuvax.com](mailto:support@accuvax.com)

## Loading a Partial Multi-Dose Vial

When to load partial: After removing or dispensing a FREEZER COVID-19 vaccine

- Load vaccine as new
- Scan FRDIGE vaccine for the correct manufacturer
- Manually enter lot & expiration
- Adjust box quantity on the card to “1”
- “Load Partial Vial”
  - Enter original pierce date
  - Enter remaining doses
- Complete load

## Do's and Don'ts – AccuVax & AccuShelf COVID19

### Do's

- Scan original packaging to store in FRIDGE
- Dispense from FRIDGE
- Reload only FRIDGE multidose vials
- Scan correct barcode to store in freezer or ultra low freezer

### Don'ts

- Do not reload FROZEN vaccines into AccuVax
- Do not scan wrong manufacturer barcode
- Do not store ultra low freezer vaccine in AccuVax

**Store in FREEZER (-15°C)**

COVID-19 vaccine: **Pfizer**

Store **PFIZER** in **FREEZER**

- Scan barcode
- Confirm “FREEZER” in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in freezer



**Store in REFRIGERATOR (2-8°C)**

COVID-19 vaccine: **Moderna (10-dose)**

Store **MODERNA (10-dose)** in  
**FRIDGE**

- Scan barcode
- Confirm “FRIDGE” in name
- Manually enter lot & expiration
- Complete load or receive
- Place in AccuVax or store in fridge

