

AccuVax Administrator Competency Checklist

Facility Name:		Date:
Administrator:		
Trainer:		
		Initials
General	Knows where to locate unit information and phone number to reach support.	
	Understands how to add a new and existing user.	
	Understands the difference between Administrator, Loading Clinician, and Clinician level of access.	
	Knows where to check for internet and portal connection.	
	Knows to be constantly up to date on Expired, Expiring, and Unknowns.	
	Capable of accessing Wi-Fi settings.	
Loading	Knows difference between 1D and 2D Barcode.	
	Able to correctly scan in 1D Barcode and enter correct Lot/Exp.	
	Understands how to properly scan vaccines and edit multi-dose vial (MDV).	
	Understands MDV requires pierced date and doses remaining need to be entered when loading opened vial.	
	Able to select items manually when needed.	
	Able to properly receive a transfer.	
	Knows how to store a box in the drawer.	
	Understands how to load a box from the drawer.	
Dispensing	Knows to allow sensors to recognize vaccine when loading.	
	Understands removing all vaccines once the tray opens.	
	Understands vaccines can be scanned or card can be selected to confirm items have been removed.	
	Knows how to select the patient manually or by search by using three methods (If applicable).	
	Knows how to add a new patient (If applicable).	
	Can properly return a vaccine if dispensed incorrectly.	
	Understand how to address "Patient Safety Caution" alerts (if applicable).	
Reloading	Logs out when transaction is finished.	
	Understands how to scan item in order to return.	
	Able to select item manually when needed.	
	Understands how to spoil dose and entire vial.	
	Understands timer tracking for MDV.	
Inventory	Understands how to return a drawn syringe into the drawer.	
	Understands the different categories.	
	Knows the difference between Program inventory.	
	Understands how to find recent activity.	
	Knows how to transfer (if applicable).	
Temperature	Knows how to remove Expired and Unknown vaccines.	
	Knows how to select "Confirm Temperature" to capture the temperature recordings.	
	Understands times setup to confirm min and max temps.	
	Knows red temperature is out of range.	

Comments:

Signatures:

X _____

X _____