

Procedure for Loading ABRYSSVO™

Loading ABRYSSVO into an AccuVax® ES?

Follow the procedure outlined below to avoid mechanical damage to your AccuVax ES.

Applies to the following NDCs:

- 0069-0344-01 (package)
- 0069-0344-10 (package)
- 0069-0344-05 (package)
- 0069-0207-01 (vial)


Overview

The Pfizer ABRYSSVO vaccine is supplied in a kit that includes three components.


Vial of Lyophilized Antigen



Syringe of Sterile Diluent

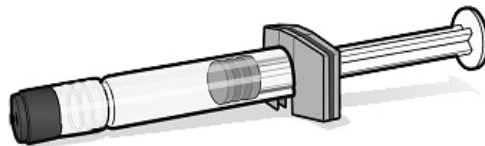

Vial Adapter

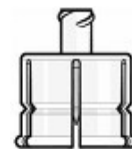

The AccuVax ES load tray cannot accommodate for storage of all three components simultaneously.

How to Safely Store ABRSYVO in AccuVax ES

1. At an AccuVax ES kiosk, log in, tap 'Load,' then select a program (.e.g., 'Private').
2. Scan the 2D barcode on a Vial of Lyophilized Antigen, or choose 'SELECT VACCINE MANUALLY.'
3. Tap 'Store diluent component in the drawer.'
4. Select 'YES, LOAD VACCINE'. The auxiliary drawer will unlock.
5. **Open the drawer, and place the Syringe(s) of Sterile Diluent and the Vial Adapter(s) in the drawer. Close the drawer, then hit next. The load door will open.**
6. **Place the vial(s)—and only the vial(s)—into the load tray. Press 'CLOSE' once you have confirmed that the vials are resting flat.**